

## iDentOz Denture Marking Kit – Directions for use

Read all Directions for Use and Safety Information before starting to mark dentures.

1. We recommend that dentures are disinfected prior to marking to avoid risk of cross-contamination.
2. Use the clean, dry abrasive pad square to abrade the surface of the denture where it is to be marked. We recommend choosing an area near the rear of the denture on the side that sits next to the cheek, as shown below. Abrade an area slightly larger than you will need to mark the dentures.



3. Ensure that the abraded area is dry and dust-free. Use the denture marking pencil to write the identifying mark required on the dentures, e.g. initials of denture wearer.
4. Apply a coat of the sealing liquid over the area that has been marked and also the area that was abraded.
5. Ensure the bottle is closed immediately to ensure longevity of sealing liquid and reduce vapours.
6. Wait for the sealing liquid to dry until it is no longer tacky (2-3 minutes) before applying another layer of sealing liquid.
7. Allow 15 minutes for the final coat of sealing liquid to dry before returning dentures.
8. Dispose of used abrasive pad. Use a clean pad for each set of dentures.

Additional information including Safety Data Sheet is available at [www.identoz.com.au](http://www.identoz.com.au)

## Safety Information

### iDentOz Denture Marking Sealing Liquid

Ingredients	Amount
Ethyl acetate	50-60%
Methyl ethyl ketone	30-40%
PMMA (non-hazardous)	10-20%

#### **DANGER**

**Highly flammable liquid and vapours**

**May cause drowsiness or dizziness**

**Causes serious eye irritation**



#### **Prevention:**

- Keep away from heat/sparks/open flame/hot surfaces – No smoking
- Keep container tightly closed
- Use only in a well ventilated area and avoid breathing vapours
- Wear protective gloves and eye protection

#### **Response:**

- In case of fire: use dry sand, dry chemical or alcohol resistant foam for extinction.
- IF ON SKIN (or hair) Rinse skin with water.
- IF INHALED: Remove victim to fresh air and keep at rest position comfortable for breathing. Call a POISON CENTRE or doctor if you feel unwell.
- IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists: Get medical advice/attention.

#### **Storage:**

- Store in a cool, well-ventilated place. Keep container tightly closed.
- Store locked up.